

Step 1 Data Belt Introduction

1 Model Number


- Model Number
 - MR1000-D(M)
 - MR1000-D(L)
- Model size
 - MR1000-D(M): 28inch ~ 36inch
 - MR1000-D(L): 36inch ~ 60inch

Warning: Recommended not to be used on underweight patients.

2 Caution

- Single use only
- (Data Belt should be used once and disposed of as skin disease may be transmittable among patients)

3 image

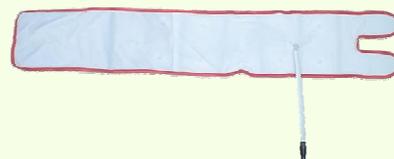


Step 2 Step-by-Step Patient Preparation

1 Align the Data belt



- Open the packaging of the Data Belt
- The Data Belt sensors should be placed on the abdomen of the patient
- The Receiver connector should be on the left side of the patient's abdomen

2 Wear the Data belt

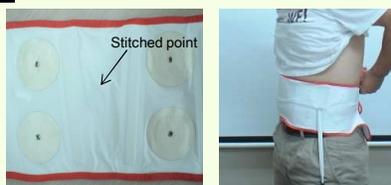

- Align the umbilical button with the stitched area between the sensor pads
 - Pull the velcro around the belt for a tight fit
- Warning:** Do not attached the Data belt above clothing

3 Attach receiver




- Turn on the MiroCam® receiver and attach the Data Belt connector

4 Remove the Capsule

- Prior to handling the capsule, the patient and nurse(s) should wash hands.
- Grasp the plastic casing covering and separate the capsule from the case.
- The capsule package should be not damaged.
- When the capsule is separated from the case, it should be blinked.
- Patients should grip the gold section of the capsule between the thumb and the forefinger of the both hands.



- Warning:** - Never touch the dome of the capsule.
- Warning:** - Do NOT use a capsule if the package is damaged.

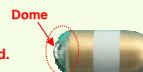

5 Check Receiver Operation

- Check the battery state. The battery indicator should be green. If it is orange, fully charge the battery.
- Check the initialization state. – The initialization indicator should be lit green. If it is orange, initialize the receiver.
- Connect the data cable to the MiroCam® receiver.
- SIG indicator will turn green after the patient holds gold bands of the capsule with hands (see step 4).

Warning: If the battery indicator on the Receiver shows that the battery is not fully charged, do not start the endoscopy procedure. The battery may be depleted during the procedure and images may not be saved properly.



6 Ingest the Capsule



- After confirming the SIG indicator is green, the patient should swallow the capsule with a small amount of water.
- Warning:** Tell Patient not to bite the capsule before ingesting. The patient should drink lots of water.

Step 3 Check Images after the Examination

1 Adjust & Wear the Receiver Bag



- Put the receiver in the bag and arrange it so that the data belt will not get tangled. Place the bag on the patient and adjust straps as necessary.

Warning: Do not allow the patient to adjust the Receiver during the procedure.

1 Upload Data

- Click the icon on the workstation to start MiroView™ Operator.
- Log in to the system as a user.
- Connect the receiver with the workstation via the USB cable.
- In MiroView™ Operator, click the icon and then click .
- Wait while the images are uploaded from the receiver.

2 Check Images

- When data uploading is completed, click the icon to open MiroView™ Client.
- Select the patient file from the list and then click the icon to review the images.