

ClutchCutter

DP2618DT

Single use Alligator Scissor Forceps




- Serrated jaws
- Rotated jaws
- Insulated outer edge

Incision, dissection and coagulation... The ClutchCutter – your 3-in-1 ESD tool.

Our rotatable alligator scissor forceps supports to perform a more efficient and safer ESD.

Features


- **Serrated jaws**
to securely grip the mucous membrane.
- **Rotating jaws**
to approach the lesion at a more precise angle.
- **Insulated outer edge**
to minimize the risk of damaging local tissue.

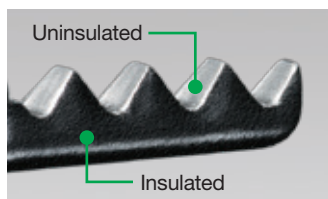


Specification

Product name	Single use Diathermic Slitter DP2618DT	
Identifier	-35-	-50-
Jaw length	3.5 mm	5.0 mm
Working length	1800 mm	
Maximum diameter of insertion portion	2.7 mm	
Forceps channel diameter of compatible endoscope	2.8 mm or more	


[Note] It may not be possible to use the ClutchCutter even if the working length and forceps channel diameter of the endoscope meet the specifications.

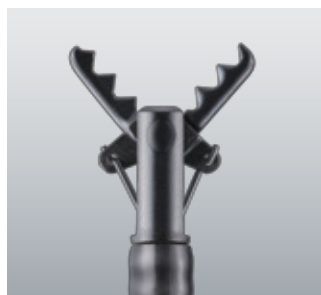

DP2618DT-35-

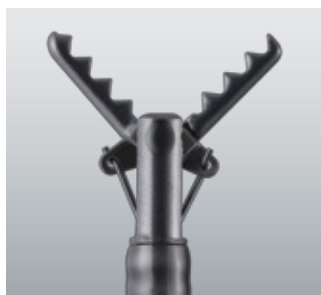

DP2618DT-50-

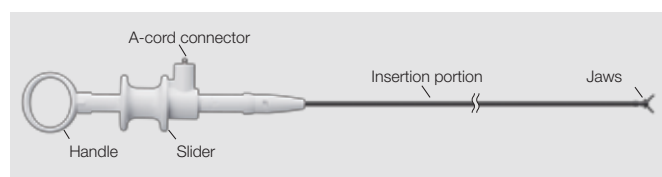

Product name: Diathermic Slitter
GMDN: 38817
Generic name: Endotherapy electro-surgical diathermy system electrode, single-use

Procedures


Incision

Grasp the mucous membrane using the serrated jaws, lift and make an incision.



Dissection

Grasp and lift the submucosa, look to confirm, and dissect.

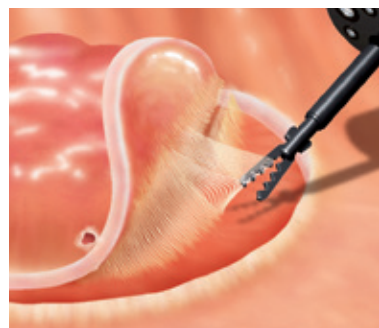

Coagulation

Coagulate any bleeding spots using the high-frequency jaws.

