



**C.R.Kennedy**  
| MEDICAL SOLUTIONS |

# VISION DISTAL CAP

The four channels and holes allow rapid fluid discharge

## Channel and Hole

Improve your endoscopic VISION  
with rapid fluid discharge

Compliant material for easy  
attachment with reduced  
scope and tissue trauma



CE 0123



### How to attach

Attach the VISION cap to the scope in the direction shown by the arrow. Avoid upside-down attachment. The attachment end is softer, thinner and has further distance to the start of the channels



Order Code	Type Name	Scope OD	Compatible Fujifilm® Endoscopes	Compatible Olympus® Endoscopes	Unit
TA02013	SHS	8.6-9.8mm	EG-760R	PCF-PH190	5/Box
TA02014	SHM	9.8-10.6mm	EG-760Z, EG-760CT, EC-740T	GIF-HQ190, GIF-1TH190, GIF-H180	5/Box
TA02015	SHL	10.6-12.6mm	EC-760R, EC-760ZP, EC-760P	PCF-H190D, PCF-H180A, GIF-2TH180	5/Box
TA02016	SHLL	12.6-14.0mm	EC-760S, EC-760Z	CF-H190, CF-HQ190	5/Box
TA02019	SHM-B (Black Cap)	9.8-10.6mm	EG-760Z, EG-760CT, EC-740T	GIF-HQ190, GIF-1TH190, GIF-H180	5/Box



Contact C.R. Kennedy for more information:

VIC 03 9823 1812 | NSW 02 9552 8341 | QLD 07 3632 6717 | SA 08 8410 8105 | WA 08 9489 8544  
medicalcustomerservice@crkennedy.com.au | crkennedy.com.au